

ADVANCIS

Activon Manuka Honey Barrier Cream

Contains 25% medical-grade Manuka honey



How does it work?

Acts as an effective moisture barrier to aid in the prevention of Moisture Associated Skin Damage (MASD), whilst also acting as a balm to moisturise skin and help reduce existing skin damage and irritation. Soothes skin and dries quickly to form a transparent barrier against bodily fluids, such as sweat, urine and stool.



How to use it?

For use on areas of vulnerable, intact and mildly irritated/damaged skin where exposure to perspiration or incontinence is an issue. Can also be used around stomas.

Ensure that the skin is clean and dry prior to application. Then use a pea sized amount of cream to cover an area approximately the size of your palm. Do NOT over-apply because a little goes a long way. Cream should dry and leave a transparent film, which is the moisture barrier.

- Do NOT use on cracked infected areas of skin
- Do NOT use on open wounds or deep puncture wounds
- Do NOT use where there are any known allergies or sensitivities to the ingredients
- Do NOT use on babies less than one year old.

Frequently Asked Questions

Q. Can Activon Manuka Honey Barrier Cream be used on broken or infected skin?

A. No, Activon Manuka Honey Barrier Cream is for intact skin that is mildly irritated, sore or damaged. Do NOT use on broken skin.

Q. Are there any age limitations?

A. Yes, do NOT use on babies less than 1 year old, as with all honey there is a risk of infant botulism.

Q. How often can the product be applied? Can the product be used at every nappy change, even if it's hourly?

A. Reapply as frequently as required or as advised by your clinical practitioner. Ensure the area is cleaned and dried before applying.

Q. Does it sting when applied?

A. No, there should not be any stinging. Some may experience a tingling sensation due to the medical grade Manuka honey working.

Please bear in mind Activon Manuka Honey Barrier Cream should only be used on intact skin, NOT on broken skin.

Q. Will it leave any residue?

A. Yes, once dried a clear moisture barrier layer is formed.

Ensure you use a pea size amount and rub in the barrier cream. Once dried, a clear barrier layer will be left behind.

