ADVANCIS

Brightwake Ltd, Lowmoor Business Park,
Kirkby in Ashfield, Nottinghamshire, NG17 7JZ
Tel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238
Email: info@advancis.co.uk
Web: www.advancis.co.uk

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ADVANCIS

ALGIVON

Alginate dressing impregnated with 100% Manuka Honey

Instructions for use

Algivon®

Alginate dressing impregnated with 100% Manuka honey

EN

Product description

Algivon is a non-adherent alginate dressing impregnated with Activon Manuka honey. It is effective against a broad spectrum of bacteria and effectively eliminates odours associated with malodorous wounds. Algivon effectively de-sloughs and de-bridges wounds and absorbs exudates whilst maintaining a moist wound healing environment and prolonging the retention of Activon Manuka honey at a wound site.

Indications

May be used on various wound types including superficial wounds, partial of full thickness wounds, sloughy wounds, necrotic wounds e.g pressure ulcers, leg ulcers, surgical wounds, burns, graft sites and infected wounds.

Directions for use

Ensure that the wound has been cleaned, gently dry the surrounding skin.

Algivon® can be placed either side down directly onto the wound bed, it can be cut to size using sharp scissors, cover with a suitable secondary dressing. For larger wounds, several dressings can be placed side by side. Depending on the tissue type within the wound bed and level of exudate your secondary dressing of choice could be a foam dressing and/or bandage. In wounds with a high level of exudate an additional highly absorbent dressing can be introduced to help manage exudate.

Frequency of change

Depending on clinical observation, dressing may be left in place for up to 7 days. Algivon will be less effective as the honey is diluted by wound exudate, to maintain an effective level, change the dressing when the colour changes significantly.

Contra-indications

Contains glucose. Monitor blood glucose levels in patients with diabetes. Do not use if allergic to bee venom/ honey. A few patients may experience pain when applied, depending on sensitivity of the wound it may be necessary to consider an appropriate level of analgesia. If pain continues unabated, discontinue use and irrigate the wound with saline solution.

Storage

Store at room temperature.

Disposal

Dispose of hygienically after use. This is a single-use medical device. Re-use of this Algivon dressing after being placed on the patient may result in patient infection and/or cross-contamination. Sterilisation, reprocessing, cleaning and/or disinfection, may also compromise the product characteristics, resulting in trauma or infection to the patient.

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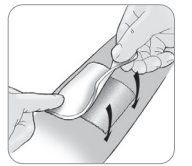
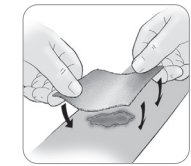
	Consult instructions for use
	Do not re-use
	Keep dry
	Sterilized using irradiation
	Keep away from sunlight
	Do not use if package is damaged
	Manufacturer
	Caution
	Authorized representative in the European Community
	Quantity
	LOT
	Batch code
	Unique Device Identifier (UDI) carrier including the AIDC and human readable information
	Medical Device
	Date of manufacture/ Country of manufacture
	CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe
	Importer
	Do not re-sterilize
	Single sterile barrier system
	Use-by date
	Indicates the authorised representative in Switzerland
	Low Exudate