ADVANCIS

 Brightwake Ltd, Lowmoor Business Park, Kirkby in Ashfield, Nottinghamshire, NG17 7JZ
Tel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238
Email: info@advancis.co.uk
Web: www.advancis.co.uk

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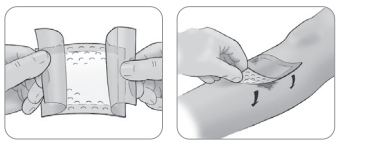
ADVANCIS

ECLYPSE ADHERENT

Super Absorbent Dressing with Soft Silicone Contact Layer

Instructions for use

Eclypse® Adherent

Superabsorbent dressing with Silfix® soft silicone contact layer

Product Description

Eclypse® Adherent is an exudate management wound dressing. The dressing combines a Silfix® soft silicone contact layer, a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-strikethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds, lymphatic legs, post-operative and dehisced wounds, fungating wounds and donor sites.

Contra-Indications

Arterial bleeds and heavily bleeding wounds.

Directions for Use

Apply to wound (white silicone face down, beige facing up) allowing a minimum of a 2cm border overlap around the wound area. Do not cut Eclypse® Adherent. Dressings can be placed side-by-side to cover large wounds. Secure the dressing pad with tape or bandage appropriately. Eclypse® Adherent may be used under compression bandages. Eclypse® Adherent can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclypse® Adherent must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

Disposal

Dispose as clinical waste.

	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using ethylene oxide		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI carrier including the AIDC and human readable information)		Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Indicates the authorised representative in Switzerland
	Device should not be cut		Moderate/High Exudate

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