Eclypse® Contour

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Superabsorbent multisite dressing

Product description

Eclipse® Contour is an exudate management wound dressing especially designed to fit to body contours. The dressing combines a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-strikethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds, lymphatic legs, post-operative and dehisced wounds, fungating wounds and donor sites.

Contra-indications

Arterial bleeds and heavily bleeding wounds.

Directions for use

Eclipse® Contour is placed white face down on the wound surface, with beige backing uppermost. For larger wounds place several dressings side by side. Protect friable skin, or drier areas of the wound with a non-adherent dressing under Eclipse® Contour. Do not cut Eclipse® Contour. Secure the dressing pad with tape or bandage appropriately. Eclipse® Contour may be used under compression bandages. Eclipse® Contour can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclipse® Contour must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

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EC REP CS Lifesciences Europe Limited
Advancis Medical is a trading name of Brightwake Ltd.