Eclypse®

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Superabsorbent dressing

Product description

Eclipse® is an exudate management wound dressing. The dressing combines a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-strikethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds, lymphatic legs, post-operative and dehisced wounds, fungating wounds and donor sites.

Contra-indications

Arterial bleeds and heavily bleeding wounds.

Directions for use

Eclipse® is placed white face down on the wound surface, with beige backing uppermost. For larger wounds place several dressings side by side. Protect friable skin, or drier areas of the wound with a non-adherent dressing under Eclipse®. Do not cut Eclipse® dressings. Secure the dressing pad with tape or bandage appropriately. Eclipse® may be used under compression bandages. Eclipse® can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclipse® must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

Warnings and Precautions

Eclipse® is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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Advancis Medical is a trading name of Brightwake Ltd.
CS Lifesciences Europe Limited