

ADVANCIS

it's about time

Advanced Wound Care

Pocket Guide

www.advancis.co.uk

Dressing Selection Guide by Indication

Product	Exudate Absorption	NECROTIC	SLOUGHY	CAVITY	GRANULATING	EPITHELIALISING	FUNGATING	INFECTED	SCARS
Recommended Primary Dressing & Wound Preparation									
Activon Tube	4	✓	✓	✓	✓	✓	✓	✓	
Activon Tulle	4				✓	✓	✓	✓	
Algivon	5	✓	✓					✓	
Algivon Plus	5	✓	✓	✓				✓	
Algivon Plus Ribbon	5	✓	✓	✓				✓	
Actilite*	4				✓	✓	✓	✓	
Advasil Conform	5								✓
Siltape	4								✓
Recommended Secondary Dressing for Use with Primary Honey Dressing*									
Eclipse Non-Backed	5	✓	✓	✓	✓	✓	✓	✓	
Eclipse	4	✓	✓	✓	✓	✓	✓	✓	
Eclipse Adherent	5	✓	✓	✓	✓	✓	✓	✓	
Eclipse Border	5	✓	✓	✓	✓	✓	✓	✓	
Advazorb	5	✓	✓	✓	✓	✓			
Advazorb Silfix	5	✓	✓	✓	✓	✓			
Advazorb Border	5	✓	✓	✓	✓	✓			
Silflex	4				✓	✓			

*Actilite should only be used with low levels of exudate

ADVANCIS

ACTIVON TUBE

100% Medical Grade Manuka honey

For use on Activon honey impregnated wound dressings or as a wound filler, directly onto the wound bed, for partial or full thickness wounds including;

- Sloughy wounds
- Pressure ulcers
- Surgical wounds
- Burns
- Graft sites
- Malodorous wounds.



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4493	20g	12	ELY864	419-7646

Suitable for Wound Types:



ADVANCIS

ACTIVON TULLE

Knitted Viscose Mesh Dressing Impregnated with 100% Manuka honey

May be used on all wound types including:

- Superficial wounds, partial or full thickness wounds, e.g. cuts and abrasions
- Pressure ulcers
- Leg ulcers
- Surgical wounds
- Burns
- Infected wounds
- Graft & donor sites

May assist with debridement if used in conjunction with Activon Tube.



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR3761	5cm x 5cm	5	EJE027	304-0631
CR3658	10cm x 10cm	5	EJE028	304-0623

Suitable for Wound Types:



ADVANCIS

ACTILITE

Non-adherent viscose net dressing coated with 99% Manuka honey and 1% Manuka oil

May be used on all wound types including:

- Superficial wounds, partial or full thickness wounds, e.g. cuts and abrasions
- Pressure ulcers
- Leg ulcers
- Surgical wounds
- Burns
- Infected wounds
- Graft & donor sites

May assist with debridement if used in conjunction with Activon Tube.



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4281	5cm x 5cm	10	EJE079	379-3759
CR3849	10cm x 10cm	10	EJE042	335-4917
CR3852	10cm x 20cm	10	EJE040	335-4925
CR4366	20cm x 30cm	10	EJE163	398-8078
CR4367	30cm x 30cm	5	EJE164	398-8086
CR4368	30cm x 60cm	5	ELY869	398-8094

Suitable for Wound Types:



ADVANCIS

ALGIVON

Alginate dressing impregnated with 100% Manuka honey



May be used on various wound types including:

- Superficial wounds
- Partial or full thickness wounds
- Sloughy wounds
- Necrotic wounds
- Pressure ulcers
- Leg ulcers
- Burns
- Graft sites
- Infected wounds



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR3831	5cm x 5cm	5	ELS846	319-3711
CR3659	10cm x 10cm	5	ELS195	319-3703

Suitable for Wound Types:



ADVANCIS

ALGIVON PLUS

Reinforced alginate dressing impregnated with 100% Manuka honey



Algivon Plus may be applied to any wound but especially:

- Pressure ulcers
- Leg ulcers
- Diabetic ulcers
- Surgical wounds
- Burns
- Graft sites
- Infected wounds



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4230	5cm x 5cm	5	ELSS49	374-9496
CR4225	10cm x 10cm	5	ELSS50	374-9512

Suitable for Wound Types:



ADVANCIS

ALGIVON PLUS RIBBON & PROBE

Reinforced alginate ribbon impregnated with 100% Manuka Honey



Algivon Plus Ribbon may be applied to any wound but especially:

- Cavities
- Sinuses
- Pressure ulcers
- Leg ulcers
- Diabetic ulcers
- Surgical wounds
- Burns
- Graft sites
- Infected wounds.



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4231	2.5cm x 20cm	5	ELSS51	374-4653

Suitable for Wound Types:



ADVANCIS

ECLYPSE

Super absorbent dressing



Moderate to heavily exuding wounds
such as:

- Leg ulcers
- Pressure ulcers
- Diabetic foot ulcers
- Sloughy wounds
- Granulating wounds
- Lymphatic legs
- Post-operative sutured or dehisced wounds
- Fungating wounds
- Donor site management



ADVANCIS

ECLYPSE NON-BACKED

Non-backed super absorbent dressing



Moderate to heavily exuding wounds
such as:

- Leg ulcers
- Pressure ulcers
- Diabetic foot ulcers
- Sloughy wounds
- Granulating wounds
- Lymphatic legs
- Post-operative and dehisced wounds
- Fungating wounds
- Donor sites



Ordering Information

Stock Codes	Size	Pack Size	NHSS Code	PIP Code
CR3818	10cm x 10cm	20	EJE029	372-1057
CR3769	15cm x 15cm	20	EJE032	305-1349
CR4460	10cm x 20cm	10	EJE85006	408-7813
CR4439	20cm x 20cm	10	EJE185	408-7805
CR3743	20cm x 30cm	20	EJE030	306-1272
CR3808	60cm x 40cm	10	EJE034	232-1131

Suitable for Wound Types:



Ordering Information

Stock Codes	Size	Pack Size	PIP Code
CR4516	10cm x 10cm	20	419-7836
CR4517	10cm x 20cm	10	419-7844
CR4518	20cm x 20cm	10	419-7935
CR4519	20cm x 30cm	20	419-7927

Suitable for Wound Types:



ADVANCIS

ECLYPSE ADHERENT

Super absorbent dressing with soft silicone contact layer



Moderate to heavily exuding wounds such as:

- Leg ulcers
- Pressure ulcers
- Diabetic foot ulcers
- Sloughy wounds
- Granulating wounds
- Lymphatic legs
- Post-operative sutured or dehiscent wounds
- Fungating wounds
- Donor site management



ADVANCIS

ECLYPSE ADHERENT SACRAL

Super absorbent Sacral dressing



Moderate to heavily exuding wounds on the sacral area such as pressure ulcers.



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR3881	10cm x 10cm	10	EKH034	325-2293
CR3883	10cm x 20cm	10	EKH011	325-2269
CR3863	15cm x 15cm	10	EKH036	325-2277
CR3864	20cm x 30cm	10	EKH015	325-2285

Suitable for Wound Types:



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR3985	17cm x 19cm	10	EME043	348-2213
CR3986	22cm x 23cm	10	EME044	348-2239

Suitable for Wound Types:



ADVANCIS

ECLYPSE BORDER/OVAL

Super absorbent dressing with soft silicone contact layer



Moderate to heavily exuding wounds such as:

- Leg ulcers
- Pressure ulcers
- Diabetic foot ulcers
- Sloughy wounds
- Granulating wounds
- Lymphatic legs
- Post-operative and dehiscent wounds
- Fungating wounds
- Donor sites



ADVANCIS

ECLYPSE FOOT

Super absorbent foot wrap dressing



For moderate to heavily exuding wounds such as:

- Leg ulcers
- Pressure ulcers
- Diabetic foot ulcers



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4296	15cm x 15cm	10	EKH061	383-6897
CR4297	20cm x 30cm	10	EKH062	383-6905
CR4300	10cm x 20cm	10	EKH063	383-5881
CR4299	15cm x 20cm	10	EKH064	383-5899
CR4298	20cm x 30cm	10	EKH065	383-5907

Suitable for Wound Types:



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4219	33cm x 48cm	5	EJE086	376-5468

Suitable for Wound Types:



ADVANCIS

ECLYPSE BOOT

Super absorbent leg wrap dressing



Moderate to heavily exuding wounds such as:

- Leg ulcers
- Pressure ulcers
- Diabetic foot ulcers
- Sloughy wounds
- Granulating wounds
- Lymphatic legs




Advancis Enclipse Boot Guide

1.



Place heel on the back fold line of Section A.

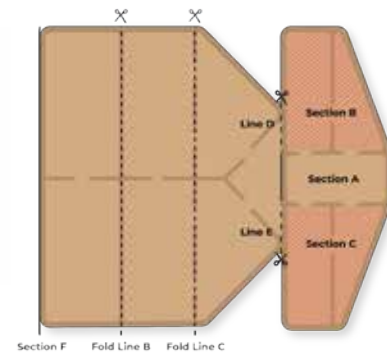

2.



3.



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4237	Small 55cm x 47cm	5	EJE085	380-4630
CR4014	Large 60cm x 70cm	5	EME042	347-6207
CR4229	X Large 71cm x 80cm	5	EJE186	408-0750

Suitable for Wound Types:



Fold Sections B and C over the top of the foot, cutting Lines D and E to make this easier if necessary. Pull up the back of the dressing towards the back of the knee (Section F).

Then wrap the remaining sides around the leg and secure with fixing tape. Can cut along Fold Lines B or C to adjust the height, if required.

ADVANCIS

ADVAZORB

Absorbent foam dressings



Suitable for a wide range of exuding wounds:

- Granulating wounds
- Leg ulcers, pressure sores
- Protection of bony prominences



ADVANCIS

ADVAZORB BORDER

Absorbent foam dressing with soft silicone wound contact layer and border



A wide range of exuding wounds including:

- Leg, foot and pressure ulcers
- Traumatic wounds
- Surgical wounds
- Abrasions and blisters.



Ordering Information

Stock Codes	Size	Pack Size	NHSC Code	PIP Code
CR4165	7.5cm x 7.5cm	10	ELA526	367-0999
CR4166	10cm x 10cm	10	ELA453	367-1005
CR4167	12.5cm x 12.5cm	10	N/A	367-1013
CR4168	15cm x 15cm	10	ELA712	367-1021
CR4169	10cm x 20cm	10	ELA1274	367-1039
CR4170	20cm x 20cm	10	ELA708	367-1047
CR4221	5cm x 5cm	10	ELA1265	374-4638

Suitable for Wound Types:



Ordering Information

Stock Codes	Size	Pack Size	NHSC Code	PIP Code
CR4190	7.5cm x 7.5cm	10	ELA547	367-0650
CR4191	10cm x 10cm	10	ELA1280	367-0668
CR4192	12.5cm x 12.5cm	10	ELA1267	367-0676
CR4193	15cm x 15cm	10	ELA550	367-0684
CR4194	10cm x 20cm	10	ELA551	367-0692
CR4195	20cm x 20cm	10	ELA552	367-0718
CR4196	10cm x 30cm	10	ELA553	367-0726

Suitable for Wound Types:



ADVANCIS

ADVAZORB SACRAL

Absorbent foam sacral dressing with film backing



For all exuding wounds on the sacral area, for example:

- Pressure ulcers



ADVANCIS

ADVAZORB HEEL

Absorbent Foam Dressing with Film Backing



Suitable for acute and chronic wounds including:

- Cuts and abrasions
- Superficial burns
- Surgical wounds
- Leg ulcers
- Pressure ulcers
- Diabetic ulcer



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4401	17cm x 19cm	10	ELA1261	401-0658
CR4402	22cm x 23cm	10	ELA1271	401-0666

Suitable for Wound Types:



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4228	17cm x 21cm	5	ELA625	374-4646

Suitable for Wound Types:



ADVANCIS

ADVAZORB AREOLA

Absorbent foam areola dressing with silfix soft silicone adhesive border



- Mastopexy
- Breast reduction
- Nipple reconstruction
- Breast augment through areola incision
- Gynaecomastia
- Other Nipple / Areola surgery



ADVANCIS

ADVAZORB SILFIX

Absorbent foam dressing with soft silicone wound contact layer



Suitable for a wide range of exuding wounds including:

- Leg Ulcers
- Foot Ulcers
- Pressure Ulcers
- Traumatic Wounds
- Surgical wounds
- Abrasions
- Blisters



Ordering Information

Stock Codes	Size	Pack Size
CR4490	13cm x 13cm	10

Suitable for Wound Types:



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR4177	7.5cm x 7.5cm	10	ELA535	367-0494
CR4178	10cm x 10cm	10	ELA536	367-0502
CR4179	12.5cm x 12.5cm	10	ELA641	367-0510
CR4180	15cm x 15cm	10	ELA538	367-0528
CR4181	10cm x 20cm	10	ELA539	367-0536
CR4182	20cm x 20cm	10	ELA540	367-0544

Suitable for Wound Types:



Products also available in Lite, See page 38

ADVANCIS

SILFLEX

Soft silicone wound contact layer

- Skin tears
- Skin abrasions
- Surgical wounds
- Lacerations
- Leg and pressure ulcers
- Topical negative pressure (primary Dressing)



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR3922	5cm x 7cm	10	EKH028	338-4385
CR3923	8cm x 10cm	10	EKH029	338-4393
CR3924	12cm x 15cm	10	EKH030	338-4401
CR3925	20cm x 30cm	10	EKH032	338-4419
CR4006	35cm x 60cm	5	EKH096	348-2205

Suitable for Wound Types:



ADVANCIS

SILTAPE

Soft silicone fixation tape

- For taping down eyelids for theatre.
- For securing fistula needles as it can be repositioned.
- For protecting friable tissue from pressure damage or sores from tubing; this is especially useful on the face and tops of ears. Siltape can be applied directly to the skin and tubing placed on top.
- For scar management



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR3938	2cm x 3m	12	EHR053	344-0336
CR3939	4cm x 1.5m	12	EHR054	344-0328

Suitable for Wound Types:



ADVANCIS

ADVASIL CONFORM

Silicone gel sheet for scar management

For the management of hypertrophic scars and keloid scars.

Advasil Conform is washable and can be used for up to 28 days once open.



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code	PIP Code
CR3845	10cm x 10cm	5	EKH006	301-1640
CR3832	10cm x 15cm	5	ELY178	329-5979

Suitable for Wound Types:



ADVANCIS

WOUND PROBE

15cm sterile wound size assessment probe

Assessment of wound size (width, length and depth) and dressing of wounds.



Ordering Information

Stock Codes	Size	Pack Size	NHSSC Code
CR4223	15cm	10	ELZ864
CR4236	15cm	50	ELZ865

Suitable for Wound Types:



Ordering Information

Medical Grade Manuka Honey & Dressings

Product	Stock Code	Size	Pack Size	NHSSC Code	Pip Code
Actilite	CR4281	5cm x 5cm	10	EJE079	379-3759
	CR3849	10cm x 10cm	10	EJE042	335-4917
	CR3852	10cm x 20cm	10	EJE040	335-4925
	CR4366	20cm x 30cm	10	EJE163	398-8078
	CR4367	30cm x 30cm	5	EJE164	398-8086
	CR4368	30cm x 60cm	5	ELY869	398-8094
Activon Tube					
	CR4493	20g	12	ELY864	419-7646
Activon Tulle	CR3761	5cm x 5cm	5	EJE027	304-0631
	CR3658	10cm x 10cm	5	EJE028	304-0623
Algivon	CR3831	5cm x 5cm	5	ELS846	319-3711
	CR3659	10cm x 10cm	5	ELS195	319-3703
Algivon Plus	CR4230	5cm x 5cm	5	ELS549	374-9496
	CR4225	10cm x 10cm	5	ELS550	374-9512
Algivon Plus Ribbon <i>(with wound probe)</i>					
	CR4231	2.5cm x 20cm	5	ELS551	374-4653

Eclypse Super absorbent Dressings

Product	Stock Code	Size	Pack Size	NHSSC Code	Pip Code
Eclypse	CR3818	10cm x 10cm	20	EJE029	372-1057
	CR4460	10cm x 20cm	10	EJE85006	408-7813
	CR3769	15cm x 15cm	20	EJE032	305-1349
	CR4439	20cm x 20cm	10	EJE185	408-7805
	CR3743	20cm x 30cm	20	EJE030	306-1272
	CR3808	60cm x 40cm	10	EJE034	232-1131

Eclypse Super absorbent Dressings

Product	Stock Code	Size	Pack Size	NHSSC Code	Pip Code
Eclypse Adherent	CR3881	10cm x 10cm	10	EKH034	325-2293
	CR3883	10cm x 20cm	10	EKH011	325-2269
	CR3863	15cm x 15cm	10	EKH036	325-2277
	CR3864	20cm x 30cm	10	EKH015	325-2285
Eclypse Adherent Sacral					
	CR3985	17cm x 19cm	10	EME043	348-2213
	CR3986	22cm x 23cm	10	EME044	348-2239
Eclypse Border					
	CR4296	15cm x 15cm	10	EKH061	383-6897
	CR4297	20cm x 30cm	10	EKH062	383-6905
Eclypse Border Oval	CR4300	10cm x 20cm	10	EKH063	383-5881
	CR4299	15cm x 20cm	10	EKH064	383-5899
	CR4298	20cm x 30cm	10	EKH065	383-5907
Eclypse Boot	CR4237	55cm x 47cm	5	EJE085	380-4630
	CR4014	60cm x 70cm	5	EME042	347-6207
	CR4229	71cm x 80cm	5	EJE186	408-0750
Eclypse Foot					
	CR4219	33cm x 48cm	5	EJE086	376-5468
Eclypse Non-Backed	CR4516	10cm x 10cm	20	EJA85033	419-7836
	CR4517	10cm x 20cm	10	EJA85031	419-7844
	CR4518	20cm x 20cm	10	EJA85032	419-7935
	CR4519	20cm x 30cm	20	EJA85030	419-7927

Advazorb Foam Dressings

Product	Stock Code	Size	Pack Size	NHSSC Code	Pip Code
Advazorb	CR4221	5cm x 5cm	10	ELA1265	374-4638
	CR4165	7.5cm x 7.5cm	10	ELA526	367-0999
	CR4166	10cm x 10cm	10	ELA453	367-1005
	CR4167	12.5cm x 12.5cm	10	N/A	367-1013
	CR4168	15cm x 15cm	10	ELA712	367-1021
	CR4169	10cm x 20cm	10	ELA1274	367-1039
	CR4170	20cm x 20cm	10	ELA708	367-1047
Advazorb Silfix	CR4177	7.5cm x 7.5cm	10	ELA535	367-0494
	CR4178	10cm x 10cm	10	ELA536	367-0502
	CR4179	12.5cm x 12.5cm	10	ELA641	367-0510
	CR4180	15cm x 15cm	10	ELA538	367-0528
	CR4181	10cm x 20cm	10	ELA539	367-0536
	CR4182	20cm x 20cm	10	ELA540	367-0544
Advazorb Border	CR4190	7.5cm x 7.5cm	10	ELA547	367-0650
	CR4191	10cm x 10cm	10	ELA1280	367-0668
	CR4192	12.5cm x 12.5cm	10	ELA1267	367-0676
	CR4193	15cm x 15cm	10	ELA550	367-0684
	CR4194	10cm x 20cm	10	ELA551	367-0692
	CR4195	20cm x 20cm	10	ELA552	367-0718
	CR4196	10cm x 30cm	10	ELA553	367-0726
Advazorb Border Lite	CR4197	7.5cm x 7.5cm	10	ELA554	367-0734
	CR4198	10cm x 10cm	10	ELA555	367-0742
	CR4199	12.5cm x 12.5cm	10	ELA556	367-0759
	CR4200	15cm x 15cm	10	ELA557	367-0767
	CR4201	10cm x 20cm	10	ELA558	367-0775
	CR4202	20cm x 20cm	10	ELA559	367-0783
	CR4203	10cm x 30cm	10	ELA560	367-0791
Advazorb Heel					
	CR4228	17cm x 21cm	5	ELA625	374-4646
Advazorb Sacral					
	CR4401	17cm x 19cm	10	ELA1261	401-0658
	CR4402	22cm x 23cm	10	ELA1271	401-0666
Advazorb Areola					
	CR4490	13cm x 13cm	10	N/A	N/A

Silflex Silicone Contact Layer Dressings

Product	Stock Code	Size	Pack Size	NHSSC Code	Pip Code
Silflex	CR3922	5cm x 7cm	10	EKH028	338-4385
	CR3923	8cm x 10cm	10	EKH029	338-4393
	CR3924	12cm x 15cm	10	EKH030	338-4401
	CR3925	20cm x 30cm	10	EKH032	338-4419
	CR4006	35cm x 60cm	5	EKH096	348-2205
Advasil Conform					
	CR3845	10cm x 10cm	5	EKH006	301-1640
	CR3832	10cm x 15cm	5	ELY178	329-5979
Siltape					
	CR3938	2cm x 3m	1	EHR053	344-0336
	CR3939	4cm x 1.5m	1	EHR054	344-0328

Wound Probe

Product	Stock Code	Size	Pack Size	NHSSC Code	Pip Code
Wound Probe	CR4223	15cm	10	ELZ864	N/A
	CR4236	15cm	50	ELZ865	N/A



Education Hub

Visit the Advancis Education Hub

A dedicated space for healthcare professionals:

- Wound care fundamentals courses
- (CPD) courses
- Wound care application and product usage tips with Tissue Viability Nurses
- Wound and product selection guides
- Wound care research and clinical updates
- Advancis product case studies and clinical evaluations

www.advancis.co.uk/education

